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EN: Instructions for use

Intended use: Thermoforming sheets for the manufacture of apparatus for dental and orthodontic treatment.

Field of application: Erkodur is exclusively suitable for the production of deep-drawn intraoral splints and devices such as bite splints, crunching splints and mini plastic splints (stabilisation), for correction splints (aligner), dressing plates, cosmetic splints, Silensor® anti-snoring splints and temporary appliances.

Fabrication, see thermoforming technique brochure and videos, www.erkodent.com

Contraindications: In case of already known allergic reaction to plastics a specific allergy to Erkodur (PETG) should be excluded.

Precautionary measures: Erkodur is only recommended for the above mentioned devices. Observe storage instructions. Attention, sensitive to humidity: Close the aluminium bag immediately.

Attention: The use is limited to the production of dental and orthodontic appliances. Use is the responsibility of the dentist. Use only on the recommendation or prescription of an authorised dentist. For single use only (deep drawing). Allergic reactions are unlikely, but cannot be excluded. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages.

Erkodur (approx. values): density 1.27 g/cm³, modulus of elasticity 2200 MPa, elongation at break, 40 %, water absorption 0.2 %, temperature resistance 63 °C

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