## Erkodur-al/-al 240

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### EN: Instructions for use

**Intended use:** Thermoforming sheets for the manufacture of apparatus for dental and orthodontic treatment.

**Field of application:** Erkodur-al is exclusively suitable for the fabrication of thermoformed intraoral splints such as aligners (correction splints) and retainers.

Fabrication, see thermoforming technique brochure and videos, [www.erkodent.com](http://www.erkodent.com)  

**Contraindications:** In case of already known allergic reaction to plastics a specific allergy to Erkodur-al (Copolyester) should be excluded.

**Precautionary measures:** Erkodur-al is only recommended for the above mentioned devices. Observe storage instructions. Attention, sensitive to humidity: Close the aluminium bag immediately.

**Attention:** The use is limited to the production of dental and orthodontic appliances. Use is the responsibility of the dentist. Use only on the recommendation or prescription of an authorised dentist. For single use only (deep drawing). Allergic reactions are unlikely, but cannot be excluded. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages.

Erkodur-al: density 1,19 g/cm<sup>3</sup>, modulus of elasticity 1462 MPa, elongation at break 179%, water absorption 0.5%, temperature resistance 85 °C

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