## Erkoflex-bleach /240

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### EN: Instructions for use

**Intended use:** Thermoforming sheets for the manufacture of apparatus for dental and orthodontic treatment.

**Field of application:** Erkoflex-bleach is thermoformed to fabricate intra-oral appliances such as bleaching trays and fluoride trays. Fabrication, see thermoforming technique brochure and videos, [www.erkodent.com](http://www.erkodent.com)  

**Contraindications:** In case of already known allergic reaction to plastics a specific allergy to Erkoflex-bleach (EVA) should be excluded.

**Precautionary measures:** Erkoflex-bleach is only recommended for the above mentioned devices. Observe storage instructions.

**Attention:** The use is limited to the production of dental and orthodontic appliances. Use is the responsibility of the dentist. Use only on the recommendation or prescription of an authorised dentist. For single use only (thermoforming). Allergic reactions are unlikely, but cannot be excluded. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages.

Erkoflex-bleach (approx. values): density 0.93 g/cm<sup>3</sup>, modulus of elasticity 70 MPa, elongation at break 450 %, water absorption 0.2 %, temperature resistance 62 °C

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