Erkoloc-pro /240/transparent/blu/green/pink

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EN: Instructions for use

Intended use: Thermoforming sheets for the manufacture of apparatus for dental and orthodontic treatment.

Field of application: Erkoloc-pro is thermoformed to fabricate comfortable intra-oral appliances such as occlusal splints, bruxism splints, correction splints (aligners), anti snoring appliances.

Fabrication, see thermoforming technique brochure and videos, www.erkodent.com  

Contraindications: In case of already known allergic reaction to plastics a specific allergy to Erkoloc-pro (PETG/TPU) should be excluded.

Precautionary measures: Erkoloc-pro is only recommended for the above mentioned devices. Observe storage instructions.

Attention, sensitive to humidity: Close the aluminium bag immediately.

Attention: The use is limited to the production of dental and orthodontic appliances. Use is the responsibility of the dentist. Use only on the recommendation or prescription of an authorised dentist. For single use only (deep drawing). Allergic reactions are unlikely, but cannot be excluded. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages.

Erkoloc-pro (approx. values): (PETG/TPU) density 1.27.12 g/cm³, modulus of elasticity 2200/27 MPa, elongation at break 40/590 %, water absorption 0.2 %, temperature resistance 63 °C

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