

# Joining ERKOFLEX foils for multi-colour mouthguards

ERKODENT AUSTRALIA Tel: 1800 242 634

[www.erkodent.com.au](http://www.erkodent.com.au)



1. Cut Erkoflex and position edges in heat protection angles (part # 660449)



2. Degrease surfaces to be joined (part #613050)



3. Allow heat gun to warm up and pass over joint 3 times or until surface is glossy.



4. When heat protection angles are removed the joint will be "hinged". Push edges together and apply pressure with one surface of a heat protection angle and allow to cool



5. Foil ready for thermoforming!