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
Highly precise condensation cured impression material on polysiloxan basis, type 0 - very high consistency, mouldable (acc. to DIN EN ISO 4823:2015)

First stage impression material (corrective impression technique), for the double mixing technique, bite fixations, forming of functional borders, etc.


Consult accompanying documents

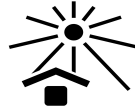Keep away from sunlight


Read instructions for use

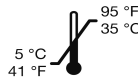Recommended storage temperature

Keep dry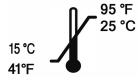

Recommended working temperature

Dosage:

To 1 level measuring spoon of **Kneton** (about 17 g) 1.5 cm of hardener paste (0.4 g) are added.

Application:

Mix **Kneton** with the hardener paste for 30 - 45 sec.

Press the material after mixing into the impression tray. Unperforated trays need to be treated before with adhesive liquid.

Leave the tray in the patient's mouth for 3.5 min. and wash off, if necessary, prepare for the corrective impression.

The impression can be filled at the earliest after 15 minutes.

**corresponds to the guide 93/42/EEC for medical devices*

Tip:

Higher room temperatures and more hardener will accelerate the settings.

Attention:

Avoid eye contact with the hardener; avoid stains on clothes.

Technical data:

- mixing can be effected manually or by machine
- mixing time: **30 – 45 sec.**
- total working time: **1.5 min.**
- time to remain in the mouth: **3.5 min.**
- setting time: **5 min.**
- max. deformation under pressure: **1 - 3 %**
- rebound after deformation: **> 99 %**
- linear contraction: **- 0.5 %**
- hardness Shore-A: **app. 60**
- disinfection possible
- galvanization possible
- recommended time to pour the cast **30 min. - 24 hours**

The user of our products is the only person responsible for the use and the application of our products. Our verbal and written advice has to be considered as advice without obligation also regarding possible protection rights of third parties.

The user is not from the obligation to check our products personally concerning suitability for the intended application. The perfect quality of our products is guaranteed according to our General Terms and Conditions.

In any case and for all damage our liability to the value of the goods supplied by us.